

COVID-19

Instructions for Replacing Plastic Laminate on HPH Face Shields



NOTE: Arrow and HPH should be oriented UP as shown.

April 1, 2020

Terminology



HPH Head frame with removable clip



Replaceable plastic laminate



HPH Face Shield

HPH Plastic Face Shield Sanitation for PPE Conservation

- HPH face shield has:
 - Head frame with removable clip that holds plastic laminate in place
 - Replaceable plastic laminate
- Write your name on the inside of the head frame. This is yours to keep and clean.
- The non-personalized plastic laminate can be cleaned with a disinfectant wipe, alcohol, or soap and water.
- Eventually it can be replaced as needed

April 1, 2020

How to Remove and Replace the Plastic Laminate on HPH Head Frame

1. Grip the head frame with thumb and forefinger as shown below. With your other hand dig under the clip as shown, gently pull off the first rivet tab. Or, push the tabs from underside to loosen the clip.



NOTE: The first removal may be a little tight/sticky when pulling off tabs; subsequent removals will be easier.

April 1, 2020

2. Slide your fingers to the 2nd rivet and gently pull off



3. Slide your fingers to the final rivet and pull clip off of frame or push from underside. The plastic laminate can now be removed from the clip.




4. Take the replacement plastic laminate with pre-punched holes. **With the arrow in the UP position**, align laminate holes over 3 rivets on removable clip



Arrow in "UP" position



5. Starting from one side, snap in rivets into head frame holes working from one side to the other

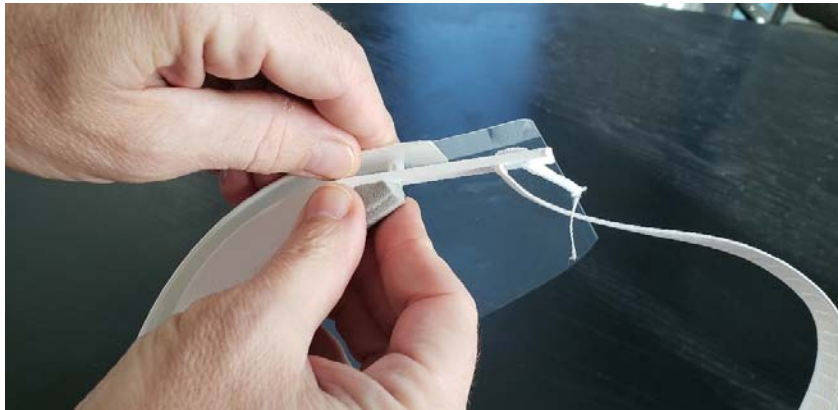

April 1, 2020