

**Surgical/Procedure Masks**

**Start of Shift:**

**N95 Masks**

Print (in ink) first and last name on the bottom edge of mask.  
**DO NOT** attach a label to ear band.



Print (in ink) first and last name on an address label.  
**DO NOT** mark the mask.



Fold label lengthwise over one of the straps on the mask.



**End of Shift:**  
Place **N95 mask** directly into the hamper marked "Used Masks".  
Discard **surgical/procedure mask** at the end of a shift or when soiled.

Hampers for N95 masks will be located at each nurse's station. EVS will collect from hampers and transport to the processing area. After UV disinfection, masks will be returned in bulk to the department from where they came for individual pick up.

N-95 masks are good for 5 disinfections or 6 uses. Masks with 4 ticks and an X should be discarded at the end of the shift.  
Note: EVS will track and mark masks after each disinfection.D

Facility specific workflows for N95 UV disinfecting process are available on the HPH COVID-19 intranet homepage.

